



Maintenance/Storage

NOTE Clean the simulator unit with warm water and soap. Do not use any chemicals or cleaners.


- 1. Flush the manikin with 1/2 liter of water and drain.

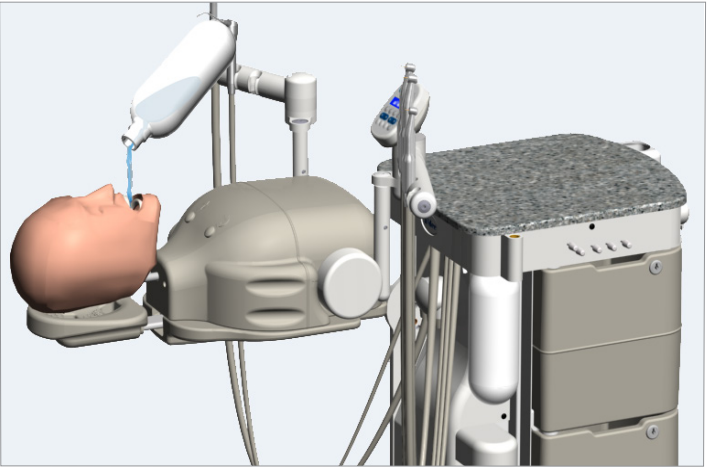


- 2. Empty the optional water bottle and optional vacuum waste jar after use.

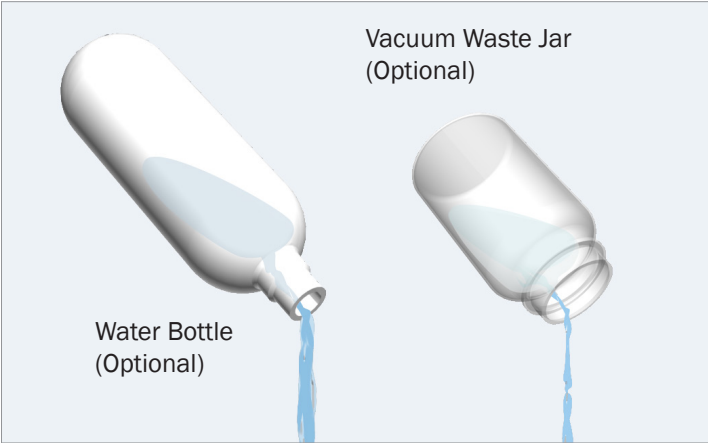

- 3. Empty the optional vacuum screen. Align the vacuum screen tab with the groove in the vacuum cannister to orient the screen properly for reinstallation.

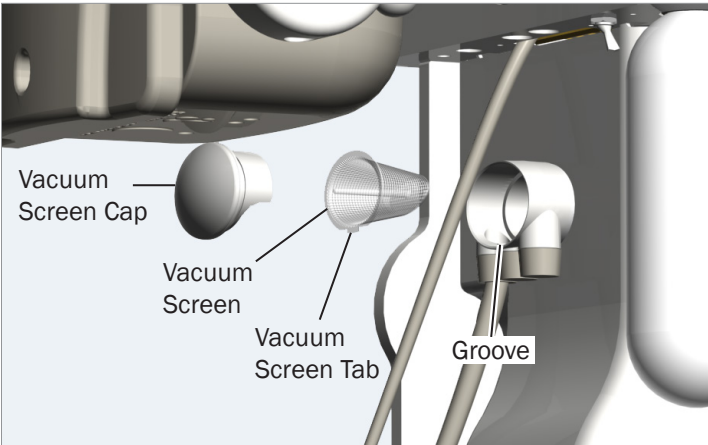

- 4. Return the unit to its storage position.

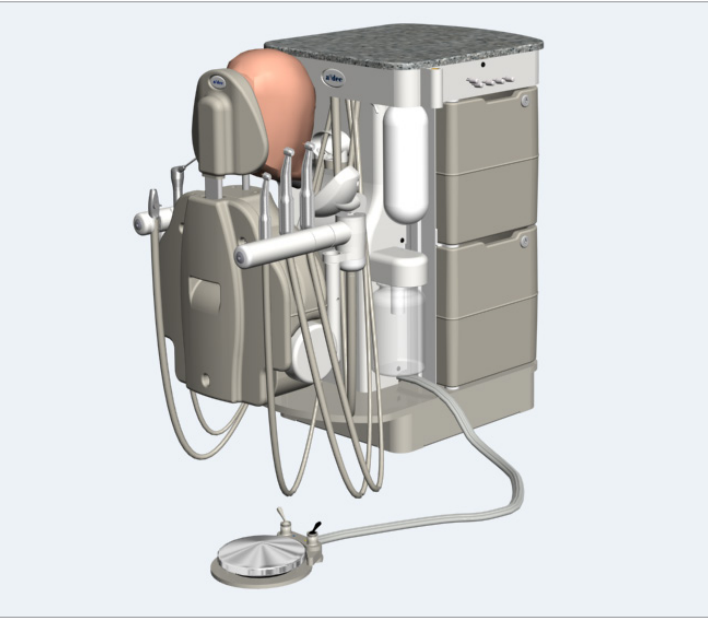

- 5. Turn the unit off.



Copyright

© 2023 A-dec, Inc. All rights reserved.

A-dec, Inc. makes no warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose. A-dec, Inc. shall not be held liable for any errors contained herein or any consequential or other damages concerning the furnishing, performance or use of this material. The information in this document is subject to change without notice. If you find any problems in the documentation, please report them to us in writing. A-dec, Inc. does not warrant that this document is error-free.

No part of this document may be copied, reproduced, altered, or transmitted in any form or by any means, electronic or mechanical, including photocopying, recording, or by any information storage and retrieval system, without prior written permission from A-dec, Inc.

Intellectual Property Rights

None of the trademarks, trade names, or proprietary symbols and icons in this document may be reproduced, copied, or manipulated in any manner without the express, written approval of the trademark owner.

A-dec, A-dec+, the A-dec design logo, and the trademarks listed at [a-dec.com/legal/trademarks](https://www.a-dec.com/legal/trademarks) are A-dec trademarks or registered trademarks in the United States and other countries. Certain touch control symbols and icons are proprietary to A-dec, Inc.



A-dec Headquarters
2601 Crestview Drive
Newberg, Oregon 97132
United States
Tel: 1.800.547.1883 within USA/CAN
Tel: +1.503.538.7478 outside USA/CAN
www.a-dec.com

A-dec China
A-dec (Hangzhou) Dental Equipment Co., Ltd.
Building 5, No. 528 Shunfeng Road
Tangqi Town, Yuhang District,
Hangzhou, Zhejiang, China 311100
Tel: 400.600.5434 within China
Tel: +86.571.89026088 outside China

A-dec Australia
Unit 8
5-9 Ricketty Street
Mascot, NSW 2020
Australia
Tel: 1.800.225.010 within AUS
Tel: +61.(0).2.8332.4000 outside AUS

A-dec United Kingdom
Austin House
11 Liberty Way
Nuneaton, Warwickshire CV11 6RZ
England
Tel: 0800.ADEC.UK (2332.85) within UK
Tel: +44.(0).24.7635.0901 outside UK

Regulatory Information and Warranty

For required regulatory information and the A-dec warranty, see the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00) available in the Resource Center at www.a-dec.com.



For quick access to this document online, scan, tap, or click this QR code, which points to:
a-dec.com/regulatory-guide.

Product Service

Product service is available through your local authorized A-dec dealer. For service information, or to locate an authorized dealer, contact A-dec at 1.800.547.1883 in the USA and Canada or +1.503.538.7478 worldwide, or visit www.a-dec.com.

Product Models and Versions Covered in This Document*

Models	Versions	Description
41L	A	Mobile Simulator
41pro	n/a	Mobile Simulator
42L	A	Stationary Simulator
42pro	n/a	Stationary Simulator

*Some products, versions, and options in this document may not be available in certain regions.



85.0016.00 Rev D
Date of Issue 2023-10-05
Copyright 2023 A-dec, Inc.
All rights reserved.



Instructions for Use

A-dec® Simulator


This product is intended for instructional use in a laboratory setting.

Setup


- 1. Turn the unit on.




- 2. Press the up/down button to raise the torso.

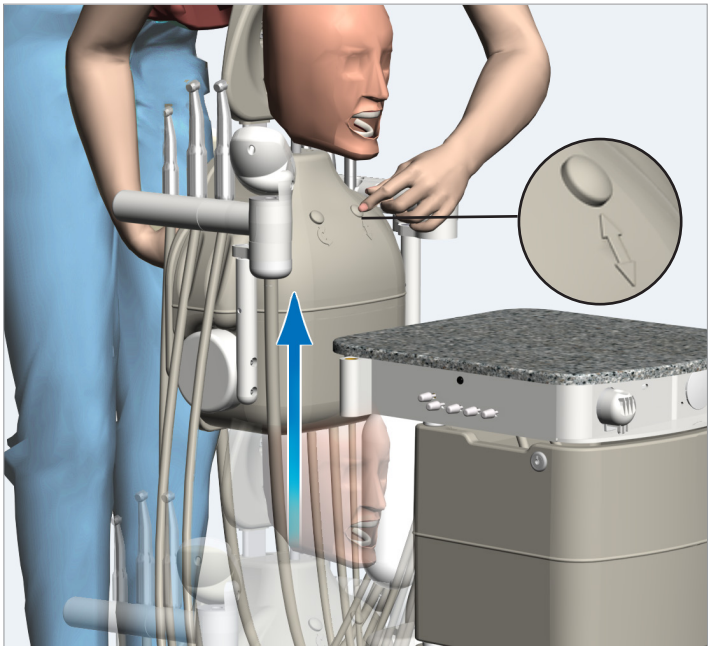

- 3. Press the rotation button to tilt the torso.

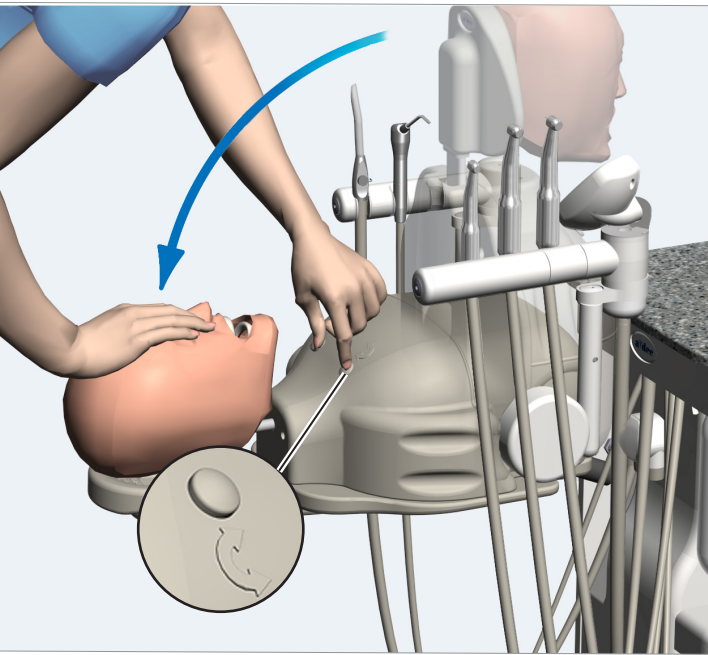

- 4. Swing the holder arms out.

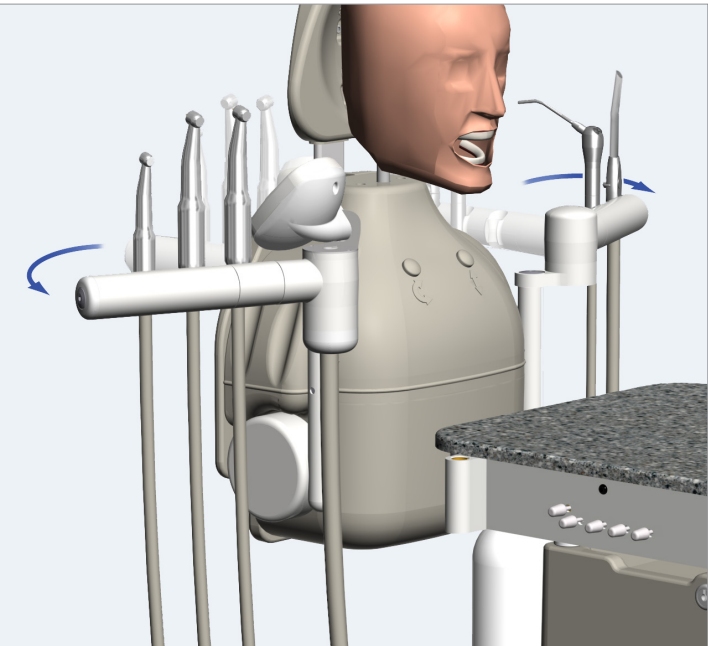


Setup (continued)

5. Fill the optional water bottle.



WARNING Use only A-dec syringe tips to prevent ejection. When installing a tip, be sure to insert the tip fully.

6. Insert the syringe tip until two clicks are felt, indicating the syringe tip is properly installed.

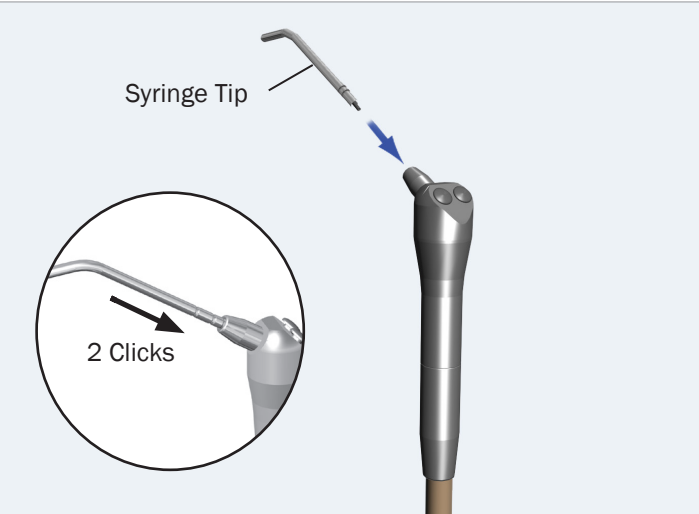

Use




CAUTION Do not turn the key all the way to the left or the stem could come out of the block.

Use the adjustment keys to adjust the air and water for handpieces.




Lift out the holders and switch sides to convert for right- or left-handed use

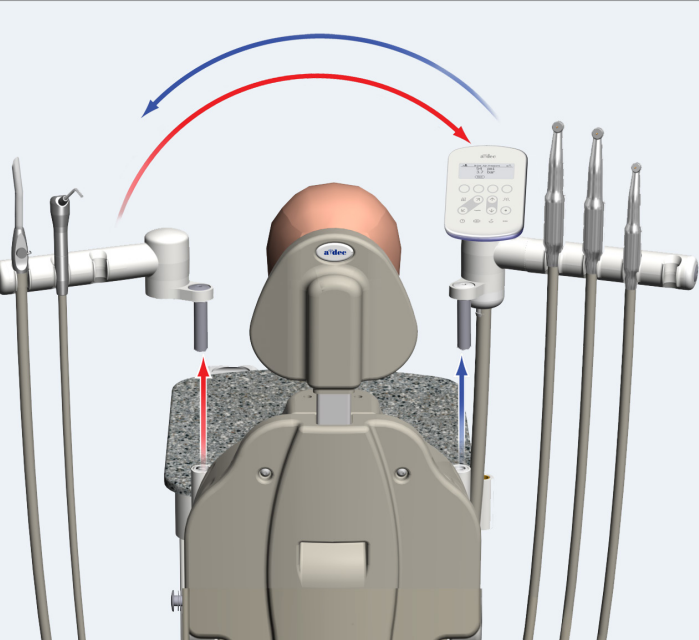


Use (continued)

Use the foot control to operate the vacuum and handpieces.

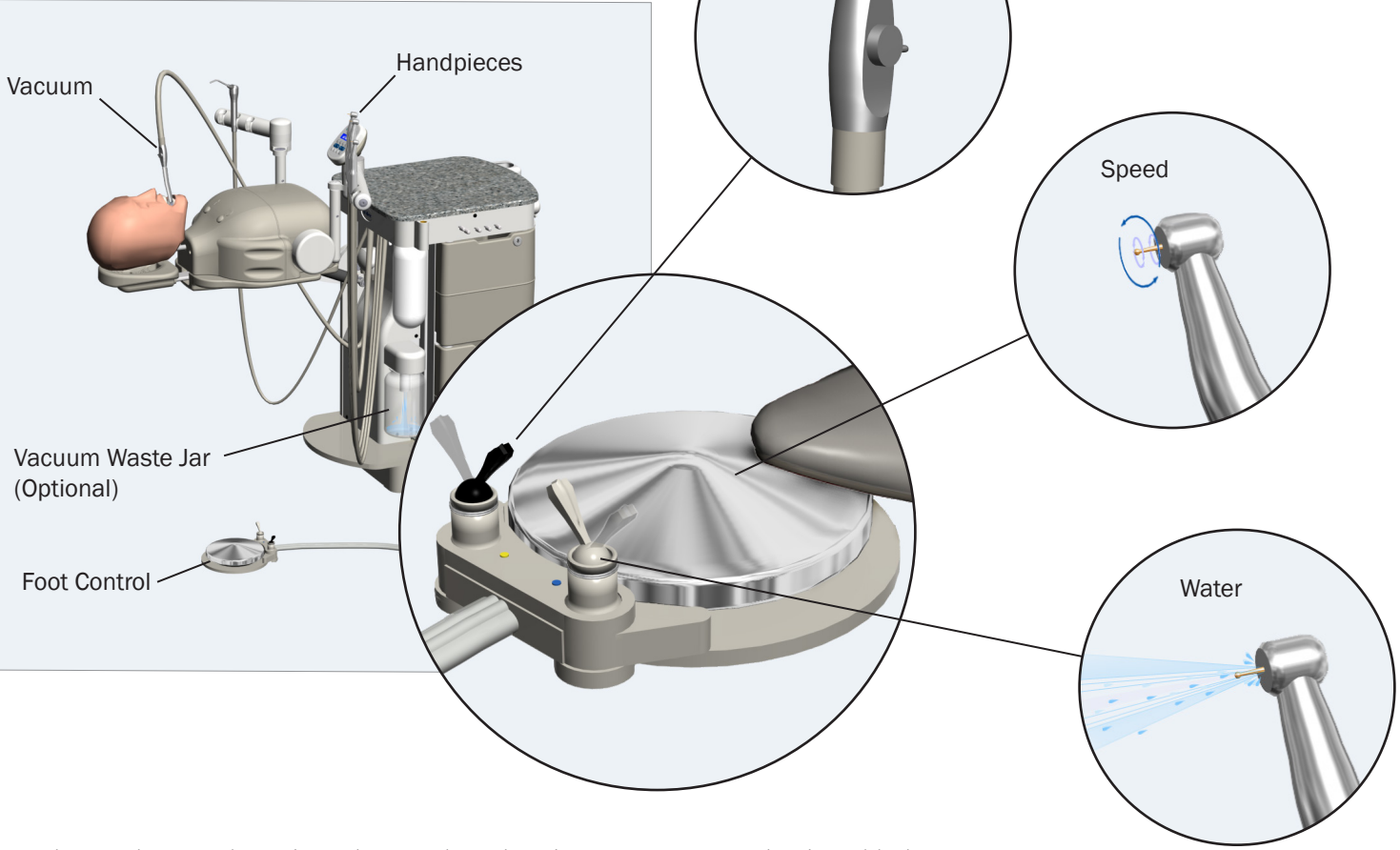


Use the touch control to adjust the simulator handpieces or operate the dental light. (Touch control is optional. Not all features shown on the touch control may be available.)

Simulator Touchpad

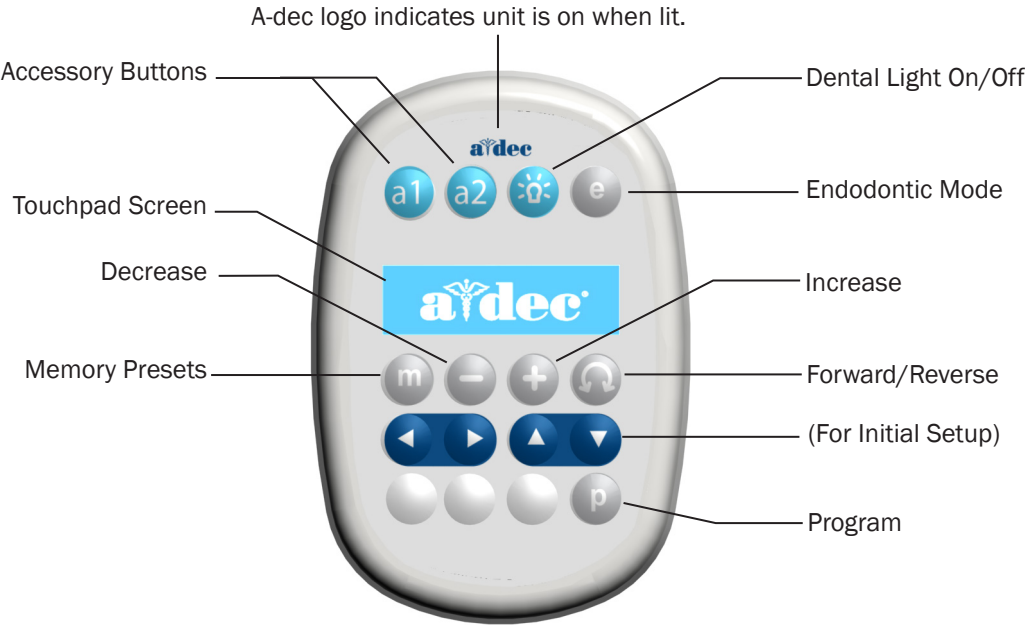

Use (continued)

CP5i Control Pad

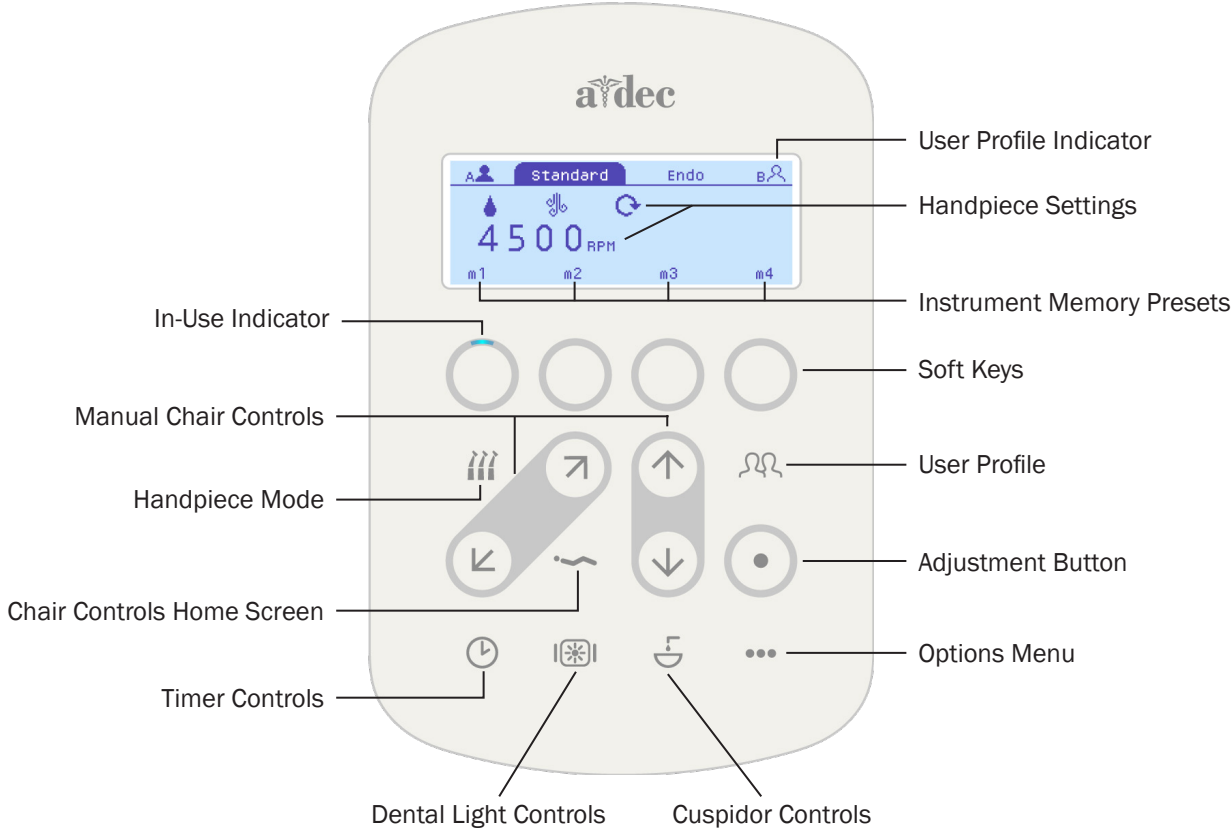

For complete CP5i control pad instructions, see the *A-dec DS7/CP5i/CP5 Touch Controls Instructions for Use* (p/n 86.0857.00) at a-dec.com/touchscreen-manual.

