

A-dec Inspire® Asepsis Quick Reference

Considerations

To preserve your investment, A-dec recommends that you use barriers where possible. Where barriers are not practical, or when cleaning and disinfection are required, use the least harmful EPA-registered disinfectants that can provide the protection you need, while reducing potential harm to the equipment.

No solution is 100 percent safe. But there are choices available today that offer a balance between effective disinfection and minimal impact to your equipment, office staff, and patients. For A-dec Inspire, use the following guidelines—and always follow the cleaner/disinfectant manufacturer's directions for use.



Least Harmful:

- ▶ Barriers
- ▶ EPA-registered disinfecting wipes* with:
 - quaternary ammonium chlorides ("*quats*") with less than 25% isopropyl alcohol
- ▶ EPA-registered disinfecting sprays* with:
 - high-dilution, water-based phenols
- ▶ Mild dishwashing liquid and water (*for cleaning only*)



Most Harmful:

- ▶ Sodium hypochlorite (*bleach*)
- ▶ Hydrogen peroxide ($pH < 4$)
- ▶ Isopropyl alcohol ($> 40\%$)
- ▶ Products with abrasives (*e.g., pumice, calcium carbonate*)
- ▶ Abrasive applicators/scrubbers



NOTE* In general, disinfecting wipes (when properly used) are less harmful than disinfecting sprays—primarily due to overspray and lack of consistent application. Always follow the disinfectant manufacturer's directions for use.



IMPORTANT For more details on cleaning and disinfecting, see the *A-dec Equipment Asepsis Guide* (p/n 85.0696.00), which is available in the Document Center at www.a-dec.com.

Surface Management — Countertops, Laminate Surfaces, and Door Infills

Surface/Item	Routine Cleaning	Light Stains	Heavy Stains/Scratches/Burns	Do NOT Use
Solid Surface 	Soft cotton cloth with a solution of mild dishwashing liquid and warm water.	Soap and water, glass cleaner, or non-abrasive household cleaners such as Fantastik® and Formula 409®.	Matte finishes only: Apply an abrasive cleaner (such as Comet® Soft Scrub®, Bon Ami®, etc.), then buff with a purple Scotch-Brite® pad using a circular motion.	Aggressive or oxidizing chemicals. See front page for other harmful disinfectants.
Quartz 	Soft cotton cloth with a solution of mild dishwashing liquid and warm water.	Soap and water, glass cleaner, or non-abrasive household cleaners such as Fantastik and Formula 409.	Contact A-dec customer service.	Aggressive or oxidizing chemicals, abrasive, strong alkaline, or highly acidic cleaners (such as bleach, Comet, Soft Scrub, SOS®, products with pumice, paint removers, furniture strippers, and tarnish, silver, or oven cleaners). See front page for other harmful disinfectants.
Laminate Surfaces 	Soft cotton cloth with a solution of mild dishwashing liquid and warm water.	Soap and water, or non-abrasive household cleaner/detergent and a soft bristled brush.	Use a soft bristled brush with a paste of baking soda and water. Lightly scrub for 10-20 seconds.	Aggressive or oxidizing chemicals, bleaching agents, or cleaners containing lye, steel wool, or other abrasives. See front page for other harmful disinfectants.
Infills 	Soft cotton cloth with a solution of mild dishwashing liquid and warm water.	Soap and water, glass cleaner, or non-abrasive household cleaners such as Fantastik and Formula 409.	Contact A-dec customer service.	Aggressive or oxidizing chemicals. See front page for other harmful disinfectants.
Control Center 	Use standard 4" x 6" barrier film (blue or clear). Standard touchscreen wipes or cleaner may be used to clean the screen. Use a disinfectant wipe when the barrier is compromised. Be sure to prevent liquid from dripping into the seam and housing.	n/a	n/a	Cleaners or disinfectants with a spray applicator. See front page for other harmful disinfectants.



A-dec Inc.
2601 Crestview Drive, Newberg, Oregon 97132
1.800.547.1883 / www.a-dec.com



For complete instructions
and video tutorials, go to:
www.a-dec.com/InspireSupport.



86.0632.00 Rev A
Copyright 2016 A-dec Inc.
All rights reserved.