

A-dec® Flush Dock

INSTRUCTIONS FOR USE



Intended Use

A hands-free accessory to assist in the dental waterline flush process.

Features

- Handpiece Tubing Holders
- Syringe Holders
- Container
- Removable Top



NOTE Flush all handpiece tubing (air and waterlines) in accordance with the regulations in your area. If no regulations exist, flush the tubing for a minimum of 20-30 seconds at the beginning of the day and after each patient.




WARNING Do not use with a vacuum system. Do not use with an HVE or saliva ejector (SE).

Use the Flush Dock When Flushing Handpieces

1. Place the Flush Dock on a flat, stable surface within reach of the air/water syringes and handpieces to be flushed.
2. Remove the syringe tips, handpieces, and handpiece motors.
3. Insert the syringes into the holders with the buttons facing the water symbol. Syringe flushing will begin automatically upon insertion.
4. Insert the handpiece tubing into the designated holders on the Flush Dock.
5. Start the manual or timed flush cycle from the touch control or use the manual toggle to flush for the required duration.
6. When the flush cycle ends, remove all syringes and handpiece tubing from the holders.
7. Remove the Flush Dock top and empty the container into a sink.

Clean and Disinfect

After the last use of the day, use a mild detergent, warm water, and soft nylon bristled brush to clean the Flush Dock. Wipe down or let air dry before storing.

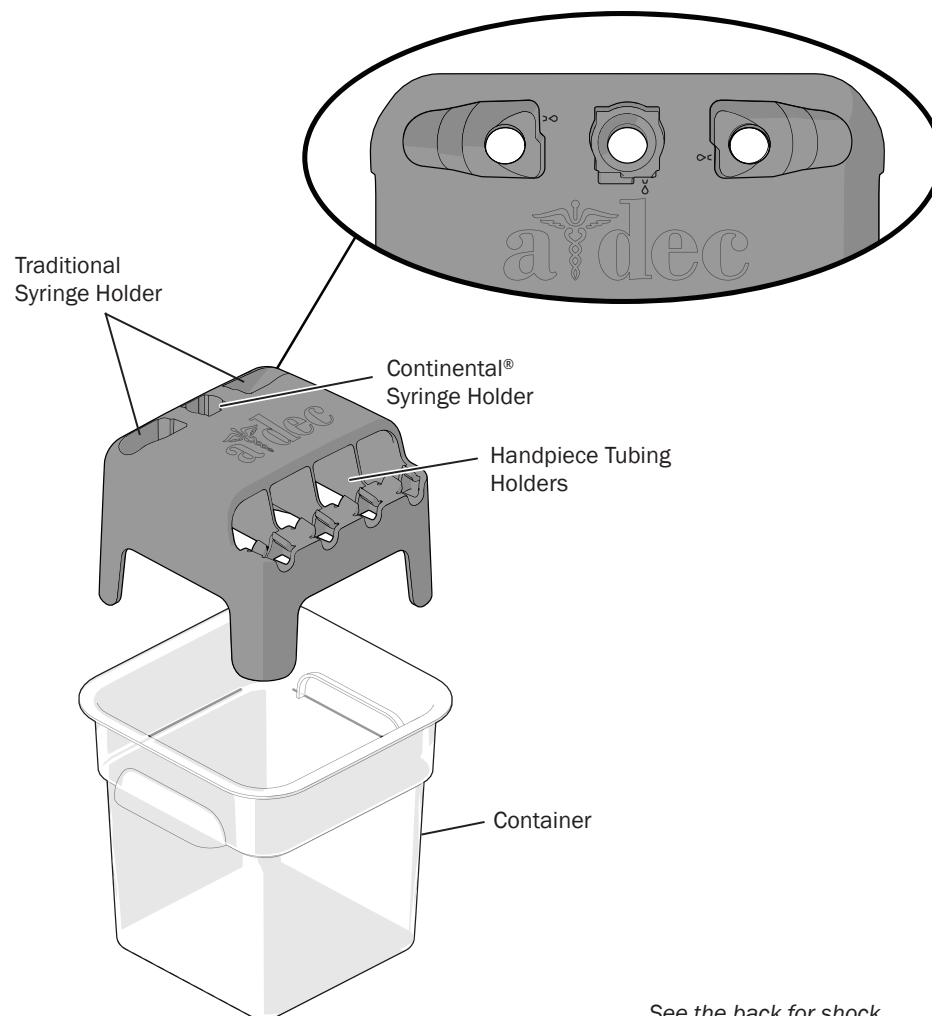

See the back for shock treatment instructions. →



WARNING Do not connect to HVE or saliva ejector (SE) lines. Do not pour purged shock solution down a cuspidor drain connected to a vacuum system.

Use the Flush Dock When Shocking with A-dec ICX Renew®*

1. Follow steps 1-4 on page 1.
2. Use the Flush Dock to hold the syringes and handpiece tubings while the shock solution is first run through the system.




NOTE Syringes flush automatically when inserted and should not remain in the holders during shock treatment dwell time.

3. When the solution reaches the tubing ends, remove the syringes from the Flush Dock.
4. Leave the handpiece tubing in the Flush Dock or return it to the unit during the shock treatment dwell time.
5. When it's time to flush the shock treatment out of the system, reinsert all syringes and handpiece tubing into the Flush Dock.
6. Follow the ICX Renew* Instructions for Use to run the flush cycle.
7. Remove the Flush Dock top and empty the container into a sink.

Clean and Disinfect

After the last use of the day, use a mild detergent, warm water, and soft nylon bristled brush to clean the Flush Dock. Wipe down or let air dry before storing.

* Note. ICX Renew may be sold under different names in different regions.

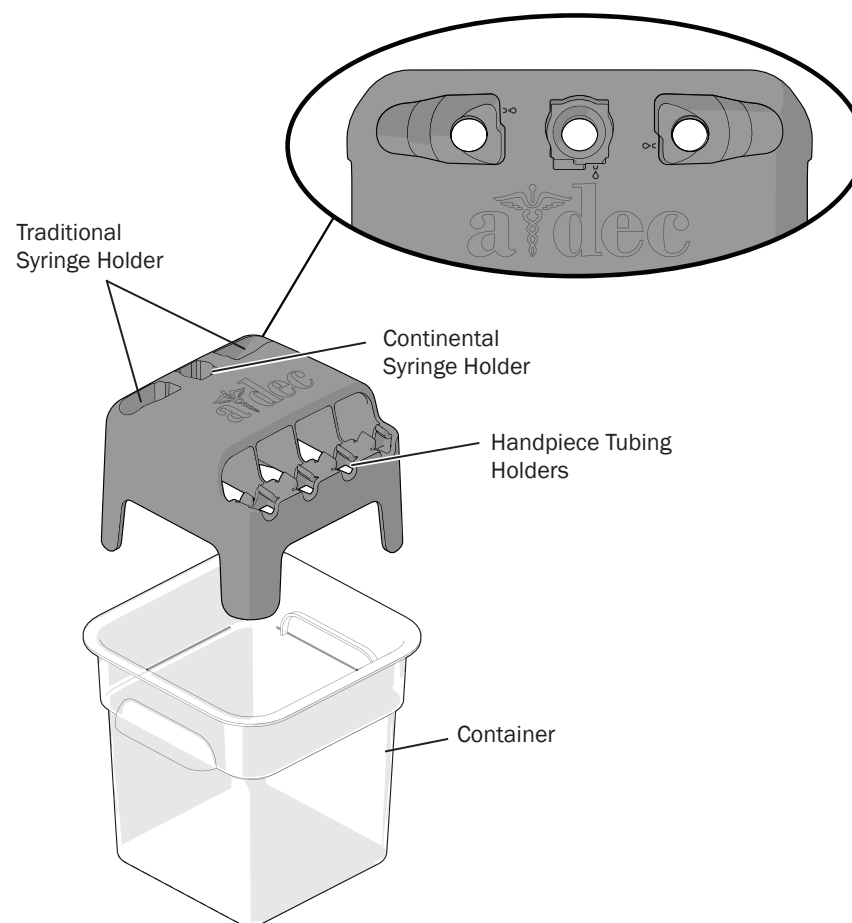

IMPORTANT For complete details on how to maintain good water quality, see the appropriate Instructions for Use provided with your A-dec waterline maintenance products.



For quick access to these documents online, scan, tap, or click this QR code, which points to: a-dec.com/waterline-manuals.