

# A-dec<sup>®</sup> 321 Doctor's Dental Stool

Instructions for Use



 a dec<sup>®</sup>

## Copyright

© 2026 A-dec, Inc. All rights reserved.

A-dec, Inc. makes no warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose. A-dec, Inc. shall not be held liable for any errors contained herein or any consequential or other damages concerning the furnishing, performance or use of this material. The information in this document is subject to change without notice. If you find any problems in the documentation, please report them to us in writing. A-dec, Inc. does not warrant that this document is error-free.

No part of this document may be copied, reproduced, altered, or transmitted in any form or by any means, electronic or mechanical, including photocopying, recording, or by any information storage and retrieval system, without prior written permission from A-dec, Inc.

## Intellectual Property Rights

None of the trademarks, trade names, or proprietary symbols and icons in this document may be reproduced, copied, or manipulated in any manner without the express, written approval of the trademark owner.

A-dec, A-dec+, the A-dec design logo, and the trademarks listed at [a-dec.com/legal/trademarks](https://www.a-dec.com/legal/trademarks) are A-dec trademarks or registered trademarks in the United States and other countries. Certain touch control symbols and icons are proprietary to A-dec, Inc.

## Regulatory Information and Warranty

For required regulatory information and the A-dec warranty, see the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00) available in the Resource Center at [www.a-dec.com](https://www.a-dec.com).



For quick access to this document online, scan, tap, or click this QR code, which points to: [a-dec.com/regulatory-guide](https://www.a-dec.com/regulatory-guide).

## Product Service

Product service is available through your local authorized A-dec dealer. For service information, or to locate an authorized dealer, contact A-dec at 1.800.547.1883 in the USA and Canada or +1.503.538.7478 worldwide, or visit [www.a-dec.com](https://www.a-dec.com).

## Product Models and Versions Covered in This Document\*

Models	Versions	Description
321	A	Dental Stool

\* Equipment shown in this document may include optional features or accessories. Some products, versions, and options in this document may not be available in certain regions.

# Features/Specifications




Item	Description
1	<b>Backrest Height-Adjustment Knob</b> – Turn counterclockwise to loosen and adjust the backrest height. -
2	<b>Backrest and Seat Tilt Paddle</b> – Lift to adjust backrest and seat tilt.
3	<b>Seat Height Paddle</b> – Lift to adjust the seat height.

Item	Specification
Seat Depth	16.5" (420 mm)
Seat Width	18.5" (470 mm)
Seat Height Adjustment	16.3 – 20.1" (410 – 500 mm) short cylinder 19.6 – 25" (500 – 640 mm) tall cylinder
Backrest Range Adjustment	20°

# Clean/Maintain

## Protect with Barriers

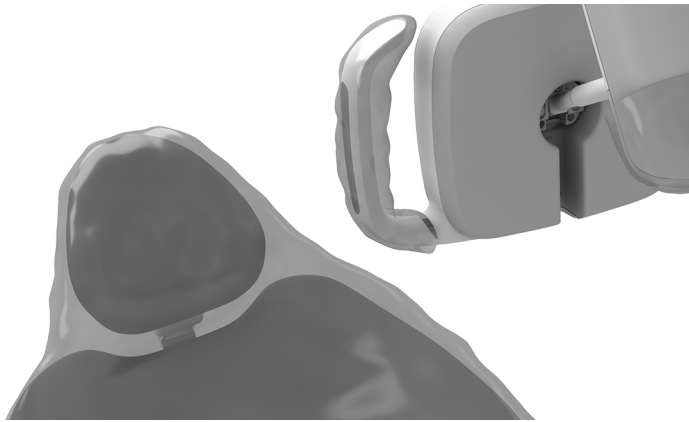

**NOTE** For proper use and disposal of barriers, see the barrier manufacturer's Instructions for Use.

A-dec recommends barrier protection for all applicable touch and transfer surfaces. Touch surfaces are areas that come into contact with hands and become potential cross-contamination points during dental procedures. Transfer surfaces are areas that come into contact with instruments and other inanimate objects.

In the USA, barriers must be produced under the Current Good Manufacturing Practice (CGMP) as specified by the U.S. Food and Drug Administration (USFDA). For regions outside the USA, refer to the medical device regulations specific to your location.



**IMPORTANT** For recommendations on cleaning and chemical disinfection of touch and transfer surfaces (where barrier protection is not applicable or when barriers are compromised), please see the *A-dec Equipment Asepsis Guide* (p/n 85.0696.00).

## Extend the Life of the Upholstery



**CAUTION** Do not use household bleach (sodium hypochlorite) or other products containing chlorine, isopropyl alcohol (greater than 25 percent by volume), or hydrogen peroxide to clean or disinfect the upholstery. It can cause rapid deterioration and damage the product.

To preserve the quality of your A-dec upholstery, use barrier covers instead of relying on chemicals. Barriers significantly extend the life of the upholstery and will help preserve its luxurious look and soft feel. To clean the upholstery, use a solution of mild dishwashing liquid and water. Use surface disinfectants only when the barriers have been compromised or there is visible spatter on the upholstery. For more information, see the *Upholstery Maintenance Guide* (p/n 86.0501.00) available in the Resource Center at [www.a-dec.com](http://www.a-dec.com).









To access translations, scan the QR code or manually enter the following URL into your web browser:

[a-dec.com/resource-center?collection=dental-stool-user-manuals](https://a-dec.com/resource-center?collection=dental-stool-user-manuals)



 **A-dec Headquarters**

2601 Crestview Drive  
Newberg, Oregon 97132  
United States  
Tel: 1.800.547.1883 within USA/CAN  
Tel: +1.503.538.7478 outside USA/CAN  
[www.a-dec.com](https://www.a-dec.com)

**A-dec Australia**

Unit 8, 5-9 Ricketty Street  
Mascot, NSW 2020  
Australia  
Tel: 1.800.225.010 within AUS  
Tel: +61.(0).2.8332.4000 outside AUS

**A-dec United Kingdom**

Austin House, 11 Liberty Way  
Nuneaton, Warwickshire CV11 6RZ  
England  
Tel: 0800.ADEC.UK (2332.85) within UK  
Tel: +44.(0).24.7635.0901 outside UK



86.1235.00 Rev A  
Date of Issue 2026-02-10  
Copyright 2026 A-dec, Inc.  
All rights reserved.

IFUnewcov5