

Ask A-dec® about...

ICX Renew®

DENTAL UNIT WATERLINE SHOCK TREATMENT

Waterline Shock Treatment

Fast-acting liquid shock solution removes odor and foul-tasting bacteria in dental unit waterlines.

Just mix the two solutions together into an empty water bottle, run through the waterlines, and soak overnight. The blue color lets you know that the solution is in the waterlines. After completing the shock treatment, flush the waterlines with water and resume your regular use.

The Benefits of ICX Renew

- Penetrates and removes the build-up of odor and foul-tasting microbial contamination bacteria.
- Will not corrode or clog waterlines.
- Blue color lets you know the solution is hard at work in the waterlines.
- Non-corrosive and gentle on equipment.
- Odorless formula.
- Recommended by A-dec, for A-dec equipment.




When used in conjunction with regular testing and adherence to protocols, A-dec ICX and ICX Renew help simplify your waterline maintenance routine.

Product description	Applications	A-dec part number
ICX Renew - Case	36	90.1691.00
ICX Renew - Box	9	90.1691.01

Unwanted guests in your waterlines?




1
Turn off delivery system,
remove and empty
the water bottle.

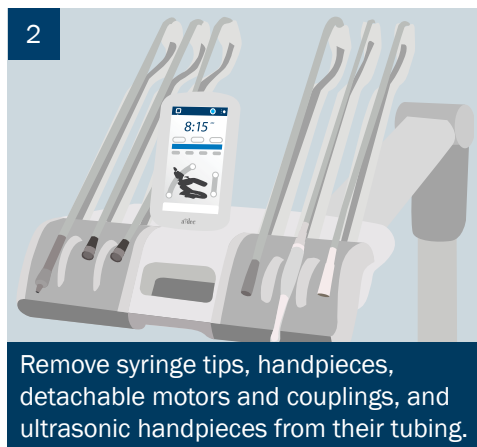

2
Remove syringe tips, handpieces,
detachable motors and couplings, and
ultrasonic handpieces from their tubing.




3
Add the contents of both ICX Renew
bottles (solutions 1 & 2) and swirl to
let the product coat all surfaces.

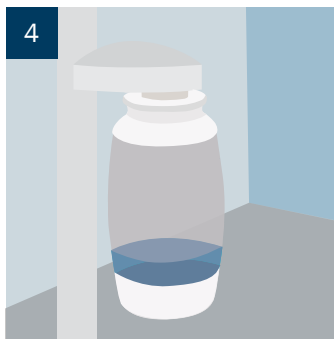


4
Re-attach water bottle and
turn delivery system on.



5
Run solution through the Dental Unit Waterlines
(DUWL) until you start to see blue water from the
tubing and cuspidor cupfill (where relevant).

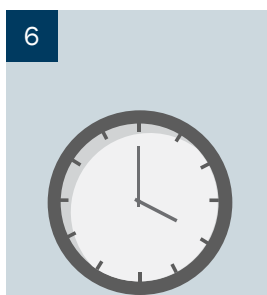

6
Let the product
sit in the DUWL
for 12-96 hours.

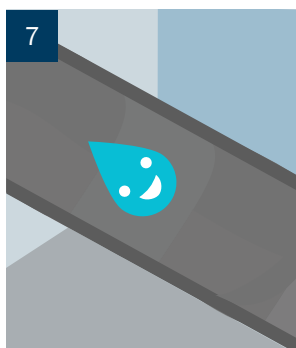

7
Turn on delivery system
and flush remaining
solution through DUWL.



8
Rinse water bottle thoroughly.
Refill bottle with water and flush
through DUWL until empty.



9
Once treatment is complete,
continue with your daily
waterline maintenance protocol.



When you **SHOCK**, let **ICX RENEW**.



Find out more about
ICX Renew Products.
a-dec.com/icx

