

A-dec 300 Pro Radius® compatibility with older A-dec chairs. (332pro, 333pro, 336pro)



Chair model	Decade 1011/1021/1221 Cascade 1040	A-dec 511A		A-dec 311B	A-dec 411	A-dec 511B	A-dec 311B, 411, 511B
Manufacture date	January 2000 or later (SN A082532)	Prior to April 2011 (SN D125124)	April 2011 to present (SN D125124)	June 22nd 2023 and prior	June 22nd 2023 and prior	June 22nd 2023 and prior	June 23rd 2023 to present
Mount	Existing mount required	Included with delivery unit (where applicable)					
Chair functionality with 300 Pro delivery	Limited functionality: Chair control	Limited functionality: Chair control Full functionality: Contact A-dec Customer Service for details					Includes full functionality
Parts required for limited functionality	1. 90.1029.02 (Chair board) - If manufactured prior to August 2005 (SN H558983) 2. Ramp/Stop Kit: - 90.1795.00 (1011/1021/1221) - 90.1796.00 (1040) Contact A-dec Customer Service for delivery ordering details	1. 43.0591.00 (DCS2-DCS1 Translator Kit) 2. 90.1918.00 (Kit, Pro/Sta to Non-UCB Chair, Adapter Cables)	43.0591.00 (DCS2-DCS1 Translator Kit)			1. 43.0591.00 (DCS2-DCS1 Translator Kit) 2. 90.1918.00 (Kit, Pro/Sta to Non-UCB Chair, Adapter Cables)	None
A-dec+ Connectivity	For A-dec+ connectivity and software updates, order A-dec+ Gateway (43.0531.00)						
Compatibility							
Delivery only (332pro/333pro/336pro)	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Dual Mount with either: 372L LED light -or- 382 monitor mounts	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Light or monitor mount only	No	No	Yes	Yes	Yes	Yes	Yes
381/482 monitor mounts	No	No	Yes	Yes	Yes	Yes	Yes

Single Delivery or Dual 300 Pro Mounting Capability



A-dec 332pro or 336pro



A-dec 333pro



A-dec 372L

OR



A-dec 382

A-dec 300 Pro Support Center mount compatibility with older A-dec chairs. (334pro/335pro on 362)


Chair model	Decade (1011/1021/1221)	Cascade 1040	Decade (1011/1021/1221)	Cascade 1040	A-dec 511A	A-dec 311B/411	A-dec 511B	A-dec 311B/411/511B
Compatible manufacture	January 2000 (SN A082532) to August 2005 (SN H558982)		August 2005 (SN H558983) or later		All years	June 22nd 2023 and prior	June 22nd 2023 and prior	June 23rd 2023 to present
Mount	Included with 362							
Chair functionality with 300 Pro delivery	Limited functionality: Chair control Full functionality: Contact A-dec Customer Service for details							Includes full functionality
Parts required for limited functionality	1. 90.1029.02 (Chair board) 2. 43.0591.00 (DCS2-DCS1 Translator Kit)		1. 43.0591.00 (DCS2-DCS1 Translator Kit)		1. 43.0591.00 (DCS2-DCS1 Translator Kit) 2. 90.1918.00 (Kit, Pro/Sta to Non-UCB Chair Adapter Cables)	1. 43.0591.00 (DCS2-DCS1 Translator Kit)	1. 43.0591.00 (DCS2-DCS1 Translator Kit) 2. 90.1918.00 (Kit, Pro/Sta to Non-UCB Chair Adapter Cables)	None
A-dec+ Connectivity	For A-dec+ connectivity and software updates, order A-dec+ Gateway (43.0531.00)							

Use the chair model and serial number to ensure the correct mounting hardware configuration during ordering.

How to read A-dec product serial numbers

12-13 Digits
Dec 2014 - Present

S/N: 14L511-A91101

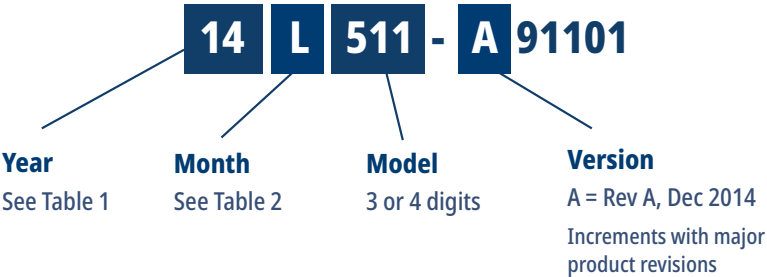

NOTE: Serial numbers may repeat every 10 years

Table 1: Years

Digits	Year		
12-13 digits or 8 digits	16 = 2016 15 = 2015	14 = 2014 13 = 2013	12 = 2012 11 = 2011
7 digits	9 = 1989, 1999 or 2009 8 = 1988, 1998 or 2008 7 = 1987, 1997 or 2007 6 = 1986, 1996 or 2006 5 = 1985, 1995 or 2005	4 = 1994 or 2004 3 = 1993 or 2003 2 = 1992 or 2002 1 = 1991, 2001 or 2011 0 = 1990, 2000 or 2010	

Table 2: Month Codes

Code	Month
A	January
B	February
C	March
D	April
E	May
F	June
G	July
H	August
I	September
J	October
K	November
L	December
M	does not indicate month (used prior to 1983)

8 Digits
June 2011 - Jan 2015



7 Digits
1985 - June 2011

